


IFU - Blood Pressure Monitor - Elemano®

Cleaning / disinfection of Main body and inflation bulb:

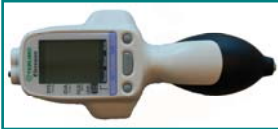

1. Cleaning of the main body / inflation bulb

Visually check the surface of the main body and the inflation bulb.

Should there be any dirt on them, gently wipe it off with a soft material, like a gauze soaked in (lukewarm) water and well squeezed. Afterwards, remove any remaining moisture with a dry piece of soft cloth.



To clean the inflation bulb (rubber bulb/tail valve), gently wipe off dirt with a soft material like a gauze soaked in (lukewarm) water and well squeezed. Afterwards, remove any remaining moisture with a dry piece of soft cloth.

2. Disinfection of the main body / inflation bulb

Only use specific disinfectants for disinfection.

Wipe the main body with a material such as a gauze soaked in disinfectant, lightly squeezed.

Remove the disinfectant with a gauze soaked in (lukewarm) water, lightly squeezed and remove any remaining moisture with a dry piece of soft cloth.

Disinfectant solutions that can be used for the main body and inflation bulb

Active ingredient:

Chlorhexidine gluconate, benzalkonium chloride

Remark: Follow the instructions (on dilution ratio etc.) of the packaging insert of each disinfectant



3. Cleaning of the inflation bulb filter

The air filter inside the inflation bulb might be clogged with dust, if the inflation bulb is unable to pressurize the cuff.

Dust is to be removed from the air filter every 6 months.

Detach the inflation bulb from the main body.

Extract the filter from the inflation bulb and lightly brush the filter with a rather soft toothbrush.

After cleaning the filter, attach the inflation bulb with the cleaned filter to the main body.



If you have any more questions concerning the Elemano® Terumo® blood pressure monitor, please check the IFU or contact your local Terumo representative.