

ADE

Operating Manual
Electronic Baby-Hanging Scale M15611



Installation and use of scale

Your ADE electronic baby hanging scale is a quality product. To achieve best weighing results strictly follow the rules of the Operating Manual.

Check carton contents

- 1 Electronic Baby-Hanging Scale
- Upper suspension
- Suspension hook
- 3 AAA batteries
- Operating Manual

Insert batteries

- Use the attached 3 AAA batteries
- Open the battery compartment on the backside of the scale by sliding it aside
- Insert batteries respecting the pole markings
- Slide the battery compartment cover back and make sure it is securely closed
- Whenever the display indicates „LO“ immediately press 'OFF' and replace batteries by fresh ones

Display Symbols

There are 3 function keys:

- **ON/OFF** key to start and shut off the scale
- **ZERO/TARE** key to set back to zero. Tare is used to deduct a certain weight (e.g. nappy)
- **UNIT** key to select weighing mode (kg/lb)

Weighing

1. To get a reliable result make sure the scale is only used in vertical position. The suspension hook for object to be weighed must remain unloaded.
2. Start the scale by pressing the **ON/OFF** key. After a warming up period and function test the display indicates '0.00'.
3. You may now weigh the baby. Make sure to use a weighing bag or weighing trouser. The material absolutely has to be a very strong fabric (like canvas) for safety reason. It must be 100% tear proof.
4. The scale shuts off automatically after 4 minutes. A manual shut off is done by using the **ON/OFF** key.
5. Make sure NOT to overload the scale. The maximum capacity is 20 kg. The scale being overloaded the display shows '**E**'. Please be advised that heavy overloading can damage the electronic part and the scale cannot be repaired. Warranty is not applicable in this case.

Tare function

Choosing this function you may weigh the baby together with different objects (e.g. nappy, vest etc.) and the weighing bag or – trouser without influencing the weight result.

- Press **ON** and wait for `0.00` indication on the display.
- Weighing bag or trouser including objects (nappy, vest etc.) to be placed on the hook.
- Press **TARE** to reset scale to zero.
- Take all objects away from the scale and place the baby incl. objects carefully in the weighing bag or trouser and hang with utmost attention on the hook. Do not swing the baby hanging in the bag.
- The scale will now display the weight of the baby only.
- After the weighing procedure make sure the baby is safely secured in its cradle.
- Press **OFF** to disconnect the scale from power.
- Additionally the scale is equipped with an automatic shut off to prolong the lifetime of the batteries.

Data-HOLD

By pressing the **HOLD** key the result displayed will be hold for 5 seconds even if the load is changing.

Change unit

Should you prefer to weigh in another weighing mode than kg you might change into another mode (lb). In this case press **UNIT** key and choose preferred mode. Make sure the scale has been started before by pressing **ON** and the displays shows `0.00`.

Cleaning and maintenance

Safety

- Your ADE baby hanging scale is a precision instrument. Handle with care. Do not drop or let heavy object fall onto it.
- When using the scale always respect the rules and advices of this Operating Manual.
- Make sure the scale is not getting wet or stored in humid places.
- Fluids penetrating the scale can severely damage the electronic part.
- **Never leave the baby hanging in the weighing trouser unattended**

Cleaning

For cleaning just use a damp cloth or common disinfection products.
Do not use any aggressive cleaning fluids.

In case of faulty results

1. Make sure batteries are inserted correctly.
2. Make sure you are in the weighing mode you selected (kg or lb).
3. Do not load the scale before the display shows `0.00`.

Warranty – Liability

ADE will repair or replace this product within a period of 2 years from date of delivery applying to defects occurred due to poor material or workmanship (presentation of invoice is necessary). All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Warranty does not include natural wear, damages caused by incorrect or negligent handling, overuse or unsuitable or improper storage. The cost of transport will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be accepted subject to using the complete original packaging and the equipment secured the way it was packed and secured originally. All the packaging should therefore be retained.

Any warranty claim will be rejected if the equipment is opened by persons not authorized expressly by ADE to do so.

Customers abroad are kindly requested to contact their local dealer for any warranty claim, customer service and spare parts.

CE Declaration of Conformity

It is hereby certified that this product meets the basic safety requirements laid down in the Directive of the Council applicable to the harmonisation of the legal provisions of the member states with regards to the electromagnetic compatibility (89/336/EEC).

Technical data

- Robust construction and housing
- Capacity: 20 kg
- Graduation: 100 g
- Dimensions: 80 x 150 x 45 mm
- Weight: approx. 500 g

Supplier

ADE GmbH & Co.
Hammer Steindamm 27 – 29
D – 22089 Hamburg
Germany

Dealer:

--